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Read the instructions before using the item and store it in a safe place for reference.

USE

Place the thermometer in an area protected from humidity and where the room temperature is between 10°C and 40°C. Beyond this range the measurements may be wrong.

Turn on the thermometer with the power button on top 30 minutes before taking the measurement. An indicating led will get activated.

Mode selection:

You can select the thermometer reading mode. To do this, insert a 3mm blunt tip into the hole at the back and press the inner button for 3 seconds, if held for longer it will not change the selection. The display will show “Sur” to measure surface temperatures, “Bod” to take people temperature or “Cou” to use it as a people counter.

People temperature taking:

Place your forehead facing the infrared reader and wait until the reading screen appears. It must be placed to a distance between 5 and 10 cm. Alternatively you can place your wrist, if so, make sure that it is uncovered of any clothing or complements such as bracelets or watches. The measurement in the wrist usually varies a few tenths regarding the measurement in the forehead.

If a fever is detected, two beeps will sound and the red alert led lights will get activated.

Fever

Abnormally low Temp

Abnormally high Temp

Scale selection:

You can select the scale in degrees °C or °F by inserting a 3mm blunt tip into the hole on the back and press the inner button. The display will show “C” for degrees Centigrade or “F” for degrees Fahrenheit.

Battery change:

Change the battery when three horizontal lines are shown

Uses 1 18650 Li-ion battery

Remove the three screws from the rear compartment.

Remove the cover and insert the new battery.

Replace the cover and screw it.

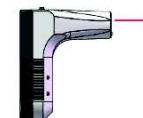
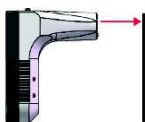
You can also to recharge the battery by plugging the supplied USB cable into common charger of a mobile phone, tablet or PC.

Computer program:

A temperature control and recording software is available.

This program will allow you to keep records of date, time, measured temperatures and number of users. This software is compatible with Windows. It is not compatible with Macintosh.

The instructions for downloading and operating the software are indicated in the attached manual.

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